

ISOLATOR™

Disposable Cotton Roll Holder

CE



2929 Northup Way
Bellevue, WA 98004 USA
425.827.4694 800.426.8051
www.pascaldental.com



EC REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands


REF

28-100



SN70145/0217

50 PACK

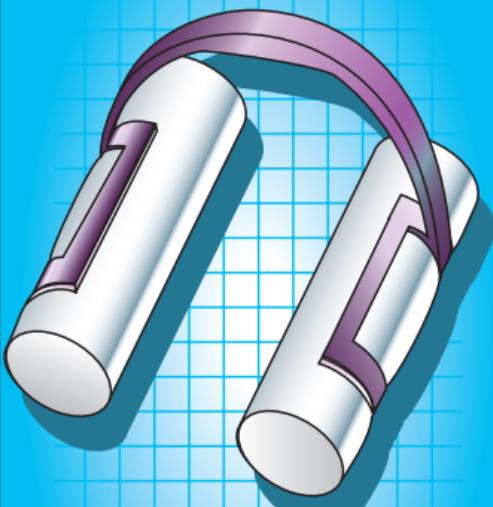

- Each clamp firmly holds two cotton rolls.
- Adaptable to cotton roll sizes 1, 2 & 3.
- Provides a dry field without restricting access or visibility
- Fully adaptable to lower or upper arches.
- Constructed of a strong, flexible nylon plastic.
- No patient discomfort
- Ideal for use with sealants
- For Professional Use Only



(01)00310866001470

ISOLATOR™

TECHNIQUE FOR LOADING "THE ISOLATOR" WITH COTTON ROLLS.

1. Firmly hold the ISOLATOR in the position illustrated, gripping the connecting loop close to the jaw being loaded



2. Use the cotton roll to pry back the jaw of the holder as illustrated.



3. Roll the cotton roll down into the open jaw.



4. Once the cotton roll is firmly positioned in the jaws of the holder, release the cotton roll, allowing the jaws to seek their original shape. The cotton roll should be firmly secured.



Caution: Reusing the Isolator puts patients at increased risk for injury or infection.