

ProPak™ Plain Cord

Gingival Retraction

Non-impregnated Cord

ProPak™ Cords are knitted retraction cord.

Contents: Each bottle contains 100 inches (254cm) Polyester/Nylon yarn.

Indications for Use: Use whenever gingival retraction is advisable in dental practice, as preceding impression taking with any elastic impression material; or prior to placement of caps, crowns, bridges; and during tooth preparation.

Contraindications: There are no known contraindications for the use of ProPak™ Cord. The components of ProPak™ Cord have a long history of safety and effectiveness for the purposes intended.

Warnings: For topical intraoral use only. Do not ingest. Keep out of reach of children.

Precautions:

- Read and follow all instructions.
- For professional in-office use only.

Amount to Use: Use as much cord of the appropriate sizes as is necessary to go all the way around the neck of each tooth to be retracted, with a slight overlap. Two loops may be packed around each tooth (see below), if sulcus space allows.

Directions for Use:

- For most effective use of ProPak™ Cord, loop a strand of thinner cord around the tooth and over the crevice.
- Trim the ends for a slight overlap, and pack the loop firmly into the sulcus.
- Apply a second loop of thicker cord over the thinner cord in the same manner.
- When ready to apply the impression material, remove the thicker cord loop only, leaving the thinner cord in place below the gingival margin until the impression procedure has been completed.
- Once the impression procedure has been completed, the thinner cord is removed also.
- Alternately, a single loop of cord will suffice.
- ProPak™ Cord completely retracts tissue within five to eight minutes and should then be removed from the mouth.

ProPak™ Cord may be saturated with astringent gels or solutions such as Pascal's Retrax Gels or Retrax Solutions prior to use for enhanced effectiveness.

Dispensed portions of cord are intended for single use only and should be disposed of immediately after use.

For product SDS information please see company website at pascaldental.com or contact Pascal directly.

ProPak™ Cord is free of gluten, soy, milk, egg, peanut, and tree nut products. ProPak™ Cord contains no artificial flavors.

For more information about Pascal's cords and latest gingival retraction materials please visit us at pascaldental.com.

**2929 Northup Way
Bellevue, WA 98004 USA
425.827.4694 | 800.426.8051
www.pascaldental.com**