

Retrax®

Hemostatic Retraction Pellets with Aluminum Sulfate

Retrax is cotton pellets saturated with a solution of aluminum sulfate and dried by a technique which retains the activity of the medication. Normal tissue moisture reactivates the medication, which promptly retracts gingival tissue and offers local hemostasis in a manner that causes no tissue damage.

Indications for Use: Use whenever gingival retraction is advisable in dental practice, as preceding impression taking with any elastic impression material; or prior to placement of caps, crowns, bridges; and during tooth preparation. Retrax will help control capillary oozing and tissue engorgement through temporary vasoconstriction. In dental use normal gingival moisture will activate Retrax by facilitating the absorption of the medication. Otherwise, the pellets may be quickly dipped in water just before being applied.

Contraindications: None known.

Precautions:

- Read and follow all instructions.
- For professional in-office use only.
- For topical intraoral use only. Do not ingest.
- Keep out of reach of children
- Do not use on patients with aluminum sulfate allergy or sensitivity.

86 °F (30 °C)



2929 Northup Way
Bellevue, WA USA 98004
425.827.4694 • 800-426-8051
www.pascaldental.com



Amount to Use: Use up to four Retrax pellets simultaneously in a single quadrant of the mouth. Retraction and hemostasis with aluminum sulfate should be limited to the equivalent of one normal quadrant of the mouth per appointment.

Directions for Use:

- Retrax can be used dry; normal tissue moisture will activate the medicament in Retrax, or they can be dipped in water prior to placement.
- Pack Retrax pellets interproximally, in conjunction with Pascal's retraction cord, to achieve best possible gingival retraction.
- When ready to apply the impression material, remove the Retrax pellets.
- Retrax completely retracts tissue within five to eight minutes and should then be removed from the mouth.

Dispensed pellets are intended for single use only and should be disposed of immediately after use.

Reusing pellets puts patients at increased risk of injury or infection.

For product SDS information please see company website at pascaldental.com or contact Pascal directly.

Pascal's Retrax pellets are free of gluten, soy, milk, egg, peanut, and tree nut products. Retrax contains no artificial flavors.

To aid in good retraction, use Pascal's hemostatic gels and solution –

- **Retrax® Gel AS – 25% Aluminum Sulfate Gel**
Makes initial cord packing easier by providing lubrication, allowing the cord to glide into the sulcus. Aluminum Sulfate aids in hemostasis.
- **Retrax® Gel AS Clear – Colorless 25% Aluminum Sulfate Gel**
Just like Pascal's Gel Cord, but made with no dye, keeping the field clear for easier visualization.
- **Retrax® Solution – 25% Aluminum Sulfate Solution**
Aids in hemostatic control. Easy to use dropper tip dispenser.

Retrax® Gel AS/Retrax® Gel AS Clear Solution are “site specific” hemostatics and are not systemic. These products will not cause blackened tissue.