

REF 15-600
RETRAX® SOLUTION
15mL



Retrax® Solution

Contents: 25% Aluminum Sulfate in a flavored aqueous base

Indications for Use:

Retrax Solution is intended to be used with all types of retraction cord and cotton pellets to aid in gingival retraction and hemostasis. Retrax Solution makes initial cord packing easier by providing lubrication when packing cord, allowing the cord to glide into the sulcus. Retrax Solution may also be used during any restoration where gingival retraction and hemostasis are indicated.

This product is for topical in-office use only. Retrax Solution is compatible with all of Pascal's medicated and plain cords and pellets.

Contraindications: None known.

Warnings: For topical intraoral use only. Do not ingest. Keep out of reach of children.

Precautions:

- For professional office use only.
- Do not use on patients with an allergy to aluminum sulfate.
- Retrax Solution contains artificial flavor; confirm that no known patient allergies exist.
- High volume suction is recommended to avoid possible ingestion.

Dosage: Use as needed to achieve and maintain tissue retraction and hemostasis.

Directions for Use:

- Soak cotton pellet or retraction cord in dispensed solution.
- Apply pellet or cord to gingival site.
- Wait 2 - 6 minutes.
- If bleeding persists, rinse area and reapply soaked pellet or cord. Wait 2 - 6 minutes.
- Remove cord or pellet.
- For best results, thoroughly rinse and dry application site.
- Take impression as usual.
- Any soaked pellets or cord should be discarded immediately after procedure.

Retrax Solution should only be applied using cotton pellets, retraction cord, or cotton tipped applicators. Solution should never be dispensed directly into patient's mouth or directly to human tissue.

For product SDS information please see company website at pascaldental.com or contact Pascal directly.

Pascal's Retrax Solution is free of gluten, soy, milk, egg, peanut and tree nut products.



2029 Northup Way
Bellevue, WA 98004 USA
425.827.4694 800.429.8051
www.pascaldental.com



CE
EMBECO EUROPE
Prinsesgracht 20
2514 AP The Hague
The Netherlands

SN70661/0919