

Siltrax AS is braided cotton cord saturated with a solution of aluminum sulfate and dried by a technique which retains the activity of the active ingredient. Normal tissue moisture reactivates the active ingredient, which promptly retracts gingival tissue in a manner that causes no tissue damage, yet fully exposes the gingival margin.

Contents: Each bottle contains 183cm braided cotton cord. The average concentration of aluminum sulfate for each size is:

Siltrax AS 7/00	0.12 mg/cm
Siltrax AS 8/0	0.189 mg/cm
Siltrax AS 9/1	0.335 mg/cm
Siltrax AS 10/2	0.57 mg/cm

Indications for Use: Use whenever gingival retraction is advisable in dental practice, as preceding impression taking with any elastic impression material; or prior to placement of caps, crowns, bridges; and during tooth preparation.

Contraindications: There are no known contraindications for the use of Siltrax AS. The chemical components of Siltrax AS have a long history of safety and effectiveness for the purposes intended.

Warnings: For topical intraoral use only. Do not ingest. Keep out of reach of children.

Precautions:

- Read and follow all instructions.
- For professional in-office use only.
- For topical intraoral use only. Do not ingest.
- Keep out of reach of children
- Do not use on patients with aluminum sulfate allergy or sensitivity.

Amount to Use: Use as much cord of the appropriate sizes as is necessary to go all the way around the neck of each tooth to be retracted, with a slight overlap. Two loops may be packed around each tooth (see below), if sulcus space allows. Retraction with an epinephrine cord should be limited to the equivalent of one normal quadrant of the mouth per appointment.

Directions for Use:

- For most effective use of Siltrax AS, loop a strand of smaller cord around the tooth and over the crevice.
- Trim the ends for a slight overlap, and pack the loop firmly into the sulcus.
- Apply a second loop of larger cord over the smaller cord in the same manner.
- When ready to apply the impression material, remove the larger cord loop only, leaving the smaller cord in place below the gingival margin until the impression procedure has been completed.
- Once the impression procedure has been completed the smaller cord is removed also.
- Alternately, a single loop of cord will suffice.
- Siltrax AS completely retracts tissue within five to eight minutes and should then be removed from the mouth.

Dispensed portions of cord are intended for single use only and should be disposed of immediately after use. Reusing dispensed portion puts patients at increased risk for injury or infection.

For product SDS information please see company website at pascaldental.com or contact Pascal directly.