



For the Deactivation of Glutaraldehyde

**Converts aldehyde waste into a
non-toxic, non-hazardous waste,
safe for regular drain disposal.**

Deactivates 24 Gallons

Directions for Use: To neutralize 1 gallon of Glutaraldehyde Solution – Add 35 grams (approximately 1 ounce by volume) of **D-Aktiv8** to the container and mix well. To neutralize 1 quart of Glutaraldehyde Solution – Add 9 grams (approximately ½ tablespoon) of **D-Aktiv8** to the container and mix well. **Wait at least 2 hours before discarding down the drain. Solution will become dark red or brown.**

Important: If Glycine is added to the container in which instruments are sterilized or disinfected, be sure to rinse container thoroughly so that all traces of Glycine are removed.

Warning: As part of good industrial and personal hygiene and safety procedures, avoid all unnecessary exposure to the chemical substance and ensure prompt removal from skin, eyes and clothing. Avoid contact with the deactivated solution. Handle the deactivated solution as you would Glutaraldehyde Solution. Refer to Glutaraldehyde Solution labeling for additional warnings, cautions, and instructions for proper handling. **For Product SDS information please see company website or contact Pascal directly.**

Net Contents: 850 grams

REF 35-725
RN 74205/0722



(01)00310866000114



2929 Northup Way
Bellevue, WA 98004
1800.426.8051
www.pascaldental.com

